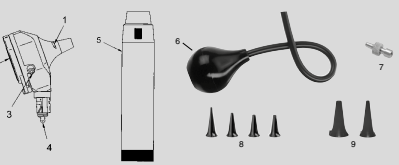
HEINE Otoloscopes

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HEINE Otoloscopes

These instructions apply to the following products of the HEINE Otoloscope series: BETA200 LED, BETA200, BETA400 LED, BETA400, K180, mini3000® F.O., mini3000® F.O. LED, BETA100, K100, mini3000®.

Please read and follow these instructions for use of and keep them for future reference.

Intended Use

HEINE otoscopes are intended for the examination of the ear, the external auditory canal and the tympanum. The instruments have optical magnifying equipment and battery or rechargeable battery powered illumination equipment. The instruments may only be used by qualified medical personnel. Patients can be examined as often as necessary without any further restrictions. The devices are intended for short term treatment and can also be used for instrumentation with non-active medical devices.

The tympanic examination can be carried out with the aid of the insufflation bulb.

After switching on the unit, a sharp drop in brightness or a flashing of the light is noticed, new batteries should be inserted or rechargeable batteries recharged.

For U.S. only:
Federal law restricts this device to sale by or on the order of a Physician or Practitioner.

Warnings and Safety Information

CAUTION! Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

NOTE! Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

Product overview (see figure)

- | | |
|---|----------------------------------|
| 1 | Insufflation bulb |
| 2 | Viewing window |
| 3 | Insufflation port |
| 4 | Connector plug |
| 5 | HEINE Handle |
| 6 | Insufflation bulb |
| 7 | Insufflation port connector |
| 8 | HEINE SANALON® S tips (re-use) |
| 9 | HEINE AIISpec® tips (single use) |

Setting up

To set up the HEINE devices, screw the instrument head into the HEINE battery handle or plug it into the HEINE rechargeable handle.

Verify that the lamp voltage complies with the supply voltage of the handle. The coloured marking on the bottom of the lamp shows you the lamp voltage:

White ring = HEINE XHL® 2.5 V bulb

Red ring = HEINE XHL® 3.5 V bulb

Black ring = HEINE LED illumination

White ring = HEINE LED illumination

Use only for use with the HEINE BETA Rechargeable Handle

This does not apply to the otoscope of the mini3000 series. Their lamp voltage is always 2.5 V independently of the handle.

Operation

The HEINE otoscope may only be inserted into the auditory canal when a tip is fitted.

Place the HEINE tip on the otoscope so that the projection inside the tip fits into the bayonet sill (1). Tighten the tip by twisting it slightly to the right.

The magnifying lens is inside the viewing window (2). To facilitate instrumentation, the viewing window can be swivelled to both sides or folded up.

The otoscopes have a connection port (3) for the optionally available insufflation bulb (6, 7).

The test of tympanic mobility can be carried out when the viewing window is closed using the insufflation bulb. Apply the pressure carefully with the insufflation bulb.

If, after switching on the unit, a sharp drop in brightness or a flashing of the light is noticed, new batteries should be inserted or rechargeable batteries recharged.

Application duration

The otoscopes are intended for short term treatment < 1 min with a break between applications of 10 minutes.

The maximum temperature at the application point at an ambient temperature of 35 °C and in continuous operation is 44.5 °C.

The setting up and operation of the HEINE handles are described in a separate instruction of use.

Hygienic Reprocessing

Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.

Classification according to KRINKO: noncritical

Spaulding Classification USA: noncritical

In the event of suspected contamination, carry out hygienic preparation of the instrument.

The reusable tips have to be reprocessed after each use.

The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.

HEINE Optotechnik only approves the reprocessing agents and procedures named in these instructions for use.

Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge.

Observe the instructions of the manufacturer of the reprocessing media.

Do not use spray or immersion disinfection, dripping wet or heavily foaming tissues. Do not use automated or ultrasonic reprocessing.

Procedure

Otoloscopes: The otoscope and the insufflation bulb must be cleaned on the outside manually with a damp cloth and inside with a cotton-wool swab (wipe cleaning and wipe disinfection).

Recommended agents

Cleaning agent: enzymatic (e.g. neodisher® MediClean)

Disinfectant agent: alcoholic (e.g. Incides® N)

Reusable tips: The reusable tips must be cleaned and disinfected manually (in an immersion bath) or by machine.

Recommended agents

Cleaning agent: enzymatic (e.g. neodisher® MediClean)

Disinfection agent: alcoholic (e.g. rotasept®), agent ortho-phthalaldehyde (e.g. Cidec® OPA)

The reusable tips can be reprocessed up to 360 cycles.

Disposable tips (HEINE AIISpec®/HEINE UniSpec®) are for single use only.

Changing the light source

Verify that the lamp voltage complies with the supply voltage of the handle.

Allow the device to cool down before changing the bulb.

BETA200 LED / BETA400 LED / mini3000® F.O. LED

With these otoscopes, the LED cannot be changed.

BETA200 / BETA400 / K180 / mini3000® F.O.

- Detach the otoscope from the handle and pull the bulb out of the guide tube of the connector plug (4) holding it at the narrow collar.

- Insert the new lamp all the way into the lamp guide.

BETA100 / K100

- Remove the tip from the otoscope and unscrew the lamp. Remove the sleeve from the old lamp.

- Refit the new bulb into the socket.

- Slide the sleeve onto the new lamp as far as it will go. This reduces stray light and reflections from the inside of the specula and should always be fitted.

mini3000

With this otoscope, swivel the viewing window (2) to the side and pull out the bulb backwards. Insert the new bulb as far as possible into the socket.

Maintenance and Service

The instruments do not require maintenance or service. The expected operational life is 5 years.

General Warnings

Check the correct operation of the device before use! Do not use the device if there are visible signs of damage or the light begins to flash.

Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anesthetic environments).

This product is not allowed to enter or be used in areas with strong magnetic fields (e.g. MRI scanners).

Do not modify the device.

Use only original HEINE parts, spare parts, accessories and power sources.

Repairs shall only be carried out by qualified persons.

Do not shine light of the instrument directly into the eyes.

Check the tip before each use for rough surfaces and edges.

In order not to exceed the surface temperature of 41 °C of the application part, keep with the application duration.

General Notes

The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.

Disposal

The product must be recycled as separated electrical and electronic devices.

Please observe the relevant state-specific disposal regulations.

Electromagnetic Compatibility

Medical electric devices are subject to special precautionary measures with regard to electromagnetic compatibility (EMC). Portable and mobile high frequency communication equipment can affect medical electric devices.

This is a device in the domestic environment, this device may cause radio interference, so that it may be necessary in some cases, to take appropriate remedial measures, as e.g. orientation, new arrangement or shielding of the device or restrict the connection to the site.

The use of accessories, converters or cables other than the ones specified by HEINE may lead to increased emission and reduced electrical immunity of the medical equipment.

The device may not be stacked directly near or used directly beside other devices. If the device is to be operated in a stack or with other devices, the device should be watched to ensure it operates properly in this location.

The appendix contains following tables:

- Guidance and manufacturer's declaration – Electromagnetic immunity

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