HEINE BETA® 4 SLIM NT HEINE BETA® SLIM

HEINE Optotechnik GmbH & Co. KG
Kientalstr. 7 · 82211 Herrsching · Germany
Tel. +49 (0) 81 52 / 38 - 0
Fax +49 (0) 81 52 / 38 - 2 02
E-Mail: info@heine.com · www.heine.com
MED 4100 2015-11-23

V-200.00.122

HEINE BETA® 4 SLIM NT / BETA® SLIM

DEUTSCH

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HEINE BETA® 4 SLIM NT / BETA® SLIM

ENGLISH

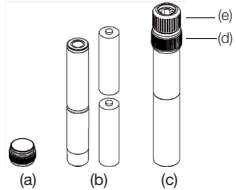
 Please read and follow these instructions for use of and keep them for future reference.

For U.S. only:
 **Federal law restricts this device to sale by or on the order of a Physician or Practitioner.**
Intended Use

The HEINE BETA® 4 SLIM NT / BETA® SLIM handle is intended exclusively for operation of medical examination instruments manufactured by the HEINE company.

Warnings and safety information
 **Caution!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow, foreground color black)
 **Note!** Note indicates valuable advice in terms of installation, operation, maintenance or repair. Notes are important, but not related to hazardous situations.

HEINE handle variants
1 BETA 4 SLIM NT Rechargeable handle X-007.99.474
2 BETA SLIM Rechargeable handle X-002.99.471
3 BETA SLIM Battery handle X-001.99.105

Setting up
 Unscrew the bottom insert (a) from the handle shell and insert the rechargeable battery in direction of the arrow – or the battery leading into the casing. Then screw the bottom insert (a) back in.
 Only charge the rechargeable handle with the following HEINE chargers:
 NT4: BETA 4 SLIM NT rechargeable handle
 NT300: BETA SLIM rechargeable handle
 Setup and operation of the HEINE chargers are described in separate instruction documents.

The automatic connector (AV)
Attaching the instrument heads
 Attach the instrument head onto the handle from above in such a way that the connector fits into the guides on the handle head. Push the instrument into the handle head until it locks in place with an audible „click“.

Removing the instrument
 To remove the instrument head turn the ring (e) to the right with the thumb and forefinger and, using the other hand, pull the instrument head up and out.

The twist-lock connector (TL)
Attaching the instrument heads
 To connect an instrument head, hold the handle in one hand, match up its mounting lugs with the recesses in the head, press in and turn the handle clockwise until securely attached.

Removing the instrument
 To remove the head rotate it anti-clockwise and pull it off.

Operation
 To switch the instrument on turn the rheostat (d) clockwise. The light intensity is indicated above the bar. 

Position „1“ shows the maximum brightness; you can set any intermediate level as desired. To turn the device off, turn the rheostat back to position „0“.

Hygienic Reprocessing
 Instructions on hygienic reprocessing must be adhered to, based on national standards, laws and guidelines.
 Classification according to KRINKO: non-critical
 Spaulding Classification USA: non-critical

 Remove this prior to processing the rechargeable battery.
 In the event of suspected contamination, carry out hygienic preparation of the instrument.
 The described cleaning and disinfection measures do not replace the specific rules applicable for the establishment.
 HEINE Optotechnik only approves the cleansing agents and procedures named in these instructions for use.
 Cleaning and disinfection may only be carried out by personnel with sufficient hygienic knowledge. Observe the instructions of the manufacturer of the reprocessing media.
 Do not use spray or immersion disinfection, dripping wet or heavily foaming tissues, automated or ultrasonic reprocessing.
 Ensure that the instrument is completely dry after reprocessing before you use it again.

Procedure
 Clean and disinfect the HEINE handle manually (wipe clean and wipe disinfect).
Recommended agents
 Cleaning agent: Neodisher® MediClean
 Disinfectant agent: alcoholic (e.g. Incides® Tissues)

Maintenance
 Routinely inspect the condition of the battery and replace them if necessary.
Changing the batteries
 (For implementation, see the relevant section)
 Unscrew the bottom insert (a) from the handle shell (c) and replace the (rechargeable) battery (b).

Service
 HEINE handles does not require regular service.

General Warnings
 Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.
 Do not use the device in fire- or explosive risk area (e.g. oxygen saturated or anaesthetic environments) This product is not allowed to be entered or used in strong magnetic fields like MRI scanners.
 Do not modify the device.
 Repairs shall only be carried out by qualified persons.
 Use only original HEINE parts, spare parts, accessories and power sources.
 Turn the handle off during the charging process.
 The handle is not designed for continuous use. The operating time of the handle depends on the maximum operating time of the examination head.

General notes
 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit www.heine.com.
 If you don't use the device for a longer period of time, please remove the batteries in advance.
 Ensure that moisture does not penetrate the area around the rechargeable battery and the bottom insert.
 Check the (rechargeable) batteries regularly. In case of any abnormalities in the operating or recharging times, replace the (rechargeable) batteries.
 If the rechargeable battery has not been recharged for a long period it may discharge automatically.

Precautions for HEINE rechargeable batteries
 Do not put into fire, do not short-circuit, do not modify, do not sterilise, do not drop, do not expose to severe shocks or to direct sunlight.
 Dispose of batteries in the event of visible or presumed damage (e.g. leaking liquid, mechanical damage). The batteries can be damaged by liquid. They must not be rinsed in running water or submerged under water.

Disposal
 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.
 Dispose at your local collection point

The appendix contains following tables
 Accessories, Technical specification, Technical specifications of the HEINE rechargeable batteries, Handle variants and components, Explanation of the used symbols

Accessories	
NT4 table charger	X-002.99.494
NT300 table charger	X-002.99.495
Adapter set (two pieces) for BETA4 SLIM NT Rechargeable handle / F.O. 4 SLIM NT Laryngoscope handle and BETA SLIM Rechargeable handle / Small F.O. Laryngoscope handle	X-000.99.086

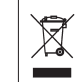
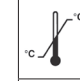
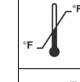
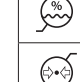
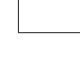
Technical specification HEINE® BETA 4 SLIM NT / BETA® SLIM	
Environmental conditions	
Operation	+10 °C to +35 °C 30 % to 90 % rel. humidity 700 hPa to 1060 hPa
Storage	+5 °C to 45°C 45% to 80% rel. humidity 500h Pa to 1060 hPa
Environmental conditions for storage	-20°C to +50°C 45% to 80% rel. humidity 500 hPa to 1060 hPa

Technical specification HEINE Rechargeable battery		
	M3Z 4 NT Li-ion	M3Z NIMH
Environmental conditions	30 % to 75 % rel. humidity 700 hPa to 1060 hPa	
Operation and charging	+10 °C to +35 °C	
Storage	+5 °C to +45 °C	
Transport	-20 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa	
Cell type	Lithium-ion technology	Nickel-metal hydride
Rated voltage [V]	3.5	3.6
Capacity [mAh]	800	1000
Charging time NT4 [h]	typ. 2.75	—
Charging time NT300 [h]	—	typ. 4

Handle variants and components			
	Handle variants	Bottom insert	Rechargeable battery/ battery
1	BETA 4 SLIM NT Rechargeable handle X-007.99.474	X-000.99.110	X-007.99.380
2	BETA SLIM Rechargeable handle X-002.99.471		X-002.99.106
3	BETA SLIM Battery handle X-001.99.105		2 x LR6/AA *

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